

DETERGENT FOR AUTOMATED ENDOSCOPE REPROCESSING

SOLUSCOPE™ ECL ENZYMATIC CLEANER

**DETERGENT FOR FLEXIBLE ENDOSCOPES. TO BE
USED EXCLUSIVELY WITH SOLUSCOPE SERIE 5 AND
SERIE XS**

- Mono-enzymatic detergent
- Proven efficacy against biofilm and proteins
- Compliant with EN ISO 15883-4
- Compatible with a large range of flexible endoscopes
- Lock key system and cannula integrated to the container



SOLUSCOPE™ ECL ENZYMATIC CLEANER

Detergent for flexible endoscopes. To be used exclusively with Soluscope Serie 5 and Serie XS



SOLUSCOPE SERIE 5



SOLUSCOPE SERIE XS

PRODUCT COMPOSITION

- Non-ionic surfactants
- Chelates
- Protease
- Excipients

PRODUCT CHARACTERISTICS

- Light yellow to amber liquid
- Density at 20°C: 1.1 g/ml
- pH pure product at 20°C: about 10.1*
- pH diluted at 0.5%: about 7.5*
- Conductivity at 0.5%: 552 µS/cm

*pH value measurements under operating conditions always by means of a pH meter with temperature compensation.

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Soluscope™ ECL Enzymatic Cleaner	2 x 5 L	3114542

INSTRUCTIONS FOR USE

- Screw the white cap connected to your Soluscope WD onto the container (refer to WD user manual)
- Shelf life after opening: 2 months while connected to the Washer-Disinfector (write the opening date on the label in the location provided for this purpose)
- Make sure that the vent hole is not blocked
- Automated dilution

STORAGE

Warning - respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products).

MEDICAL DEVICE USE SAFELY

Class I medical device. Please read label and product information.



FOR PROFESSIONAL USE ONLY.


SOLUSCOPE SAS
(An Ecolab Company)
100, rue du Fauge
Z.I Les Paluds - 13400
Aubagne - France

Distributed by:
ECOLAB EUROPE GmbH
Hofwiesenstrasse 349
CH-8050 Zürich
Switzerland

ECOLAB®

www.ecolab.com

Soluscope

www.soluscope.com