

Incidin™ OxyFoam NG

PACKAGING AND ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Incidin™ OxyFoam NG	6 x 750ml	ON REQUEST
Incidin™ OxyFoam NG	2 x 5L	ON REQUEST
Drain cock (DIN 51) for 5L Canister	1	ON REQUEST

Incidin™ OxyWipe NG

PACKAGING AND ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Incidin™ OxyWipe NG	6 x 100 wipes (20 x 20 cm)	ON REQUEST
Incidin™ OxyWipe NG XL	6 x 50 wipes (25 x 37 cm)	ON REQUEST
Wall mounted dispenser for Incidin™ OxyWipe NG (Stainless Steel)	6 x 1	100 338 77
Wall mounted holder for wipe flowpacks (durable Polypropylene)	12 x 1	100 576 86



SYNERGISTIC HYDROGEN PEROXIDE SOLUTION DISINFECTS AND CLEANS IN ONE STEP

Incidin™ OxyFoam NG & Incidin™ OxyWipe NG

READY TO USE CLEANING AND DISINFECTION SOLUTION

The advanced formulation of Incidin™ OxyFoam NG and Oxywipe NG provides the benefits of the patented* hydrogen peroxide solution combined with synergistic effect of Phenoxyethanol results in;

- Efficient formula without perfume and dye
- Cleaning and disinfection in one step, leaving the surface visibly clean
- Formulation based on biodegradable ingredients

Their features make Incidin OxyFoam NG and Oxywipe NG suitable for use on a wide range of surfaces in healthcare facilities and an ideal product for intensive care and other high-risk areas.

This enables the products to be used in practice as intended in your protocols, delivering the disinfection results and compliance level mandatory for effective disinfection to restrict the spread of pathogens.



*European Patent EP1987121

Incidin™ OxyFoam NG

READY TO USE CLEANING AND DISINFECTION SOLUTION

EFFECTIVE

- broad spectrum efficacy and short contact times combined with the properties of a gentle cleaner

SAFE

- broad material compatibility, including alcohol sensitive surfaces like ABS, PMMA plexiglas

CONVENIENT

- cleans and disinfects in one step

CHARACTERISTICS

- Dense foam for better surface adhesion (750 ml)
- Can be combined with Incidin Dry Wipes System (5L)

MICROBIOLOGICAL PROPERTIES FOR MEDICAL DEVICE AND BIOCIDES APPLICATION

EFFICACY	CONTACT TIME
Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions	
Bactericidal, yeasticidal	1 min
EN 16615* (bactericidal, yeasticidal)	1 min
EN 16615* (fungicidal)	5 min
EN 16615* (tuberculocidal/mycobactericidal)	15 min
EN 13727* (bactericidal)	1 min
EN 13624* (yeasticidal)	2 min
EN 13624* (fungicidal)	15 min
EN 14348* (tuberculocidal/mycobactericidal)	30 min
EN 14476* (enveloped virus)	5 min
EN 14476** (limited spectrum virucidal)	30 min
EN 14476** Adenovirus	30 min
EN 14476** Norovirus (MNV)	30 min
EN 14476* Polyomavirus SV 40	5 min
EN 14476* Rotavirus	5 min
Effective against enveloped viruses acc. to RKI recommendation 10/2022***	30 sec

*dirty conditions ** clean conditions *** DVV 2015 Suspension Test

MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.

APPLICATION AREA



Medical Device:

Ready-to-use surface disinfection and cleaning solution for non-invasive and non-immersable medical devices according to Directive 93/42/EEC.

Biocide:

Ready-to-use surface disinfection and cleaning solution as Biocide: for general surfaces.

INSTRUCTIONS FOR USE

For the 6x750ml bottle:

Ready-to-use solution.

Wet surface thoroughly, ensuring contact with entire surface, wipe the surface with a clean wipe and leave to dry. Or apply the product to a clean wipe, wipe the surface and leave to dry. Respect the indicated contact time and conditions for the required antimicrobial activity. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass.

Medical Device:

Rinsing is unnecessary unless when areas to be treated are intended to come in contact with skin or mucosa. Please consult manufacturer's instruction for use for cleaning and disinfection of medical devices.

For the 2x5L canister:

Ready-to-use solution to combine with Incidin Dry Wipes System (detailed instruction to prepare the dry wipes is given on the dry wipes packaging).

Wipe surface thoroughly, ensuring contact with entire surface and leave to dry. Respect the indicated contact time and conditions for the required antimicrobial activity. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass.

Medical Device:

Rinsing is unnecessary unless when areas to be treated are intended to come in contact with skin or mucosa. Please consult manufacturer's instruction for use for cleaning and disinfection of medical devices.

INGREDIENTS

Hydrogen peroxide (CAS 7722-84-1): 2.0 g/100g ;
Phenoxyethanol (CAS 122-99-6): 0.5 g/100g

STORAGE

Storage: between +5°C and +25°C.

Shelf life after first opening:

End of expiry date.



FOR PROFESSIONAL USE ONLY.

Incidin™ OxyWipe NG

READY TO USE CLEANING AND DISINFECTION WIPE

EFFECTIVE

- broad spectrum efficacy and short contact times combined with the properties of a gentle cleaner

SAFE

- broad material compatibility, including alcohol sensitive surfaces like ABS, PMMA plexiglas

CONVENIENT

- cleans and disinfects in one step

CHARACTERISTICS

- High quality, dimpled polypropylene wipe material (50 gsm) for efficient application

MICROBIOLOGICAL PROPERTIES FOR MEDICAL DEVICE AND BIOCIDES APPLICATION

EFFICACY	CONTACT TIME
Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions	
Bactericidal, yeasticidal	1 min
EN 16615* (bactericidal, yeasticidal)	1 min
EN 16615* (fungicidal)	5 min
EN 16615* (tuberculocidal/mycobactericidal)	15 min
EN 13727* (bactericidal)	1 min
EN 13624* (yeasticidal)	2 min
EN 13624* (fungicidal)	15 min
EN 14348* (tuberculocidal/mycobactericidal)	30 min
EN 14476* (enveloped virus)	5 min
EN 14476** (limited spectrum virucidal)	30 min
EN 14476** Adenovirus	30 min
EN 14476** Norovirus (MNV)	30 min
EN 14476* Polyomavirus SV 40	5 min
EN 14476* Rotavirus	5 min
Effective against enveloped viruses acc. to RKI recommendation 10/2022***	30 sec

*dirty conditions ** clean conditions *** DVV 2015 Suspension Test

MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.

APPLICATION AREA



Medical Device:

Ready-to-use surface disinfection and cleaning wipe for non-invasive and non-immersable medical devices according to Directive 93/42/EEC.

Biocide:

Ready-to-use surface disinfection and cleaning wipe as Biocide: for general surfaces.

INSTRUCTIONS FOR USE

Ready-to-use wipe.

Take single wipe. Wipe surface thoroughly, ensuring contact with entire surface and leave to dry. Respect the indicated contact time and conditions for the required antimicrobial activity. Close lid after use. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass.

Medical Device:

Rinsing is unnecessary unless when areas to be treated are intended to come in contact with skin or mucosa.

Please consult manufacturer's instruction for use for cleaning and disinfection of medical devices.

INGREDIENTS

Impregnation solution : 100g contains the following active ingredients : Hydrogen peroxide (CAS 7722-84-1): 2.0 g ; Phenoxyethanol (CAS 122-99-6): 0.5 g

STORAGE

Storage: between +5°C and +25°C.

Shelf life after first opening:

3 months.



FOR PROFESSIONAL USE ONLY.